

# Eyelid Wipes

FOR DAILY EYELID HYGIENE

ocufresh®

Preservative-free

## INDICATIONS

**Ocufresh Eyelid Wipes** is indicated for topical treatment of the eyelids and eyelashes for both adults and children, even in the presence of secretions resulting from allergies, inflammation and/or infections, such as blepharitis and blepharoconjunctivitis of various kinds. Its use is particularly indicated for the pre- and/or post-ophthalmic surgery treatment. **Ocufresh Eyelid Wipes** is also recommended for daily treatment of the periocular area in individuals who have a predisposition for the occurrence of blepharitis and blepharoconjunctivitis of various kinds. Due to its specially formulated composition, is also suitable for contact lens wearers.

## PROPERTIES

All components of **Ocufresh Eyelid Wipes** are useful to obtain optimal treatment, while fully respecting the particular physiology of the periocular area, which is particularly delicate and sensitive. The product is capable of providing accurate treatment, by mechanical removal of secretions, foreign bodies, dust, etc.

## COMPOSITION

**Soft wipe, soaked in the following solution:** Aqua, Pentylene glycol, Glycerin, Trideceth-8, Centaurea cyanus water, Euphrasia officinalis flower/leaf/stem water, Sodium chloride, Hydroxyacetophenone, PEG-40 hydrogenated castor oil, Phenoxyethanol, Disodium phosphate, Disodium EDTA, Bisabolol, Sodium phosphate.

## INSTRUCTIONS FOR USE

Having cleaned your hands carefully, open the sachet and extract the wipe. Apply the wipe to the eyelid area, wiping it delicately to allow for adequate cleansing, removal of ocular secretions, and any other types of deposit (smog, airborne particles, dust, etc.). In the case of particularly persistent secretions or the occlusion of a tear duct, it is advisable to warm the closed sachet for a few minutes in lukewarm water (approx. 40 °C) and then use the wipe as a hot compress, massaging the area delicately. On the other hand, in case of phlogosis, it is advisable to cool the sachet in the refrigerator for a few hours and then apply the open wipe to the eye, keeping it in place for a few minutes as a cold compress.

## WARNINGS

The wipe is disposable. Reusing it may spread infection. Keep in a cool and dry place. Keep out of reach of children. For external use only. Discontinue treatment in the case of evident irritation or hypersensitivity to the product. In case of infection do not use the same wipe for both eyes. Read instructions carefully. Any serious incident related to the medical device that we supply must be reported to the manufacturer and the competent authority of the Member state where it occurred.

 Promeia Italia Srl  
MIND Viale Decumano, 36  
20157 Milano (MI), Italy  
www.promeia.com

 Blumont Healthcare  
Ltd 23 Moortown Close  
Grantham - Lincs - UNITED  
KINGDOM NG31 9GG

LOT



Rev. 06/2023

Sort your waste.

BOX	SACHET
PAP 21 PAPER	C/LDPE 90 PLASTIC



**BLUMONT**

www.ocufresh.co.uk

IFU1718.00