



ocufresh®

Preservative free

Comfort Eye Drops

Singles - Carmellose Sodium 0.5% w/v

Composition:





Carboxymethylcellulose Sodium 5.0 mg, Sodium Chloride, Potassium chloride, Calcium chloride dihydrate, Magnesium chloride hexahydrate, Sodium lactate solution, Sodium hydroxide and/or hydrochloric acid to adjust pH, Water for injection.

Intended Use:

Ocufresh Comfort - Singles eye drops preservative-free is used to lubricate dry eyes.

Instructions for use:

- Wash and dry your hands thoroughly before use;
- Detach the single dose container from the strip (image 1) and open the single-dose container by twisting and pulling the tab (image 2);
- Hold the single-dose container into other hand, turn the container upside down and apply slight pressure to the single-dose container to release 1 or 2 drops of Ocufresh Comfort - Singles Eye Drops into the eye (image 3);
- Repeat the process for the other eye, if required;
- This product is not for repeat use, immediately discard the single-dose container after use (image 4);
- The product can be used as required or according to the advice of your doctor.

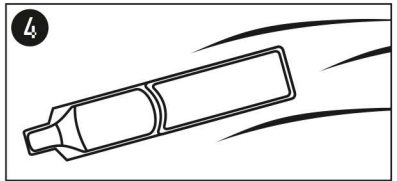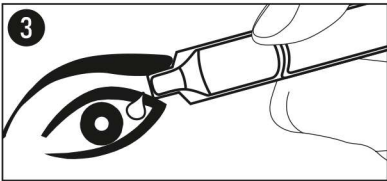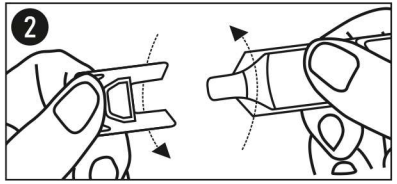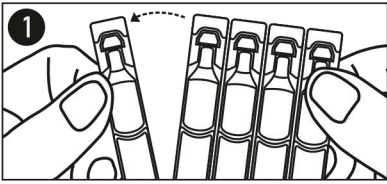

Warning and Precautions:

- This product is only for use in eyes;
- The product is intended for use only by the concerned patient;
- Do not use the product if the container is damaged when you open it for the first time;
- Do not use if the solution is cloudy or discoloured;
- Keep out of the sight and reach of children;
- Do not swallow;
- During the application of the eye drops, do not touch your eyes or any other surface with the dropper of the container to avoid any infections or damaging your eyes;
- Not for treatment of any pathological eye conditions;
- Do not re-sterilize the final packed product as it is not suitable for re-sterilization;
- The administration of other eye medicines at the same time as this product should be carried out under the supervision of your doctor who can evaluate possible interactions; there should be an interval of at least 15 minutes between the application of any other eye medicine and this product;
- Check the expiry date on the box before use; do not use the product after expiry date indicated on the package;
- After use, dispose of the product according to local regulations;
- Consult your doctor if symptoms persist.



30
Single-use
containers
0.4 ml



ocufresh[®]

Preservative free

Comfort Eye Drops

Singles - Carmellose Sodium 0.5% w/v

Possible side effects:

- As with other eye drops, this product can cause eye pain, redness or irritation of the eyes.
- If any of these effects persist or worsen or persist over 72 hours, promptly contact your doctor.

Package:

30 Single dose containers with a volume of 0.4ml.


Storage conditions:

- Store between 15°C and 25°C.
- Do not Freeze.
- Keep away from sunlight by placing the single-dose containers in the carton box.

NOT FOR INJECTION

MD

STERILE A

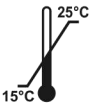

Store between
15°C and 25°C



Do not resterilise



Caution, see
instruction for use



Do not use if
package is damaged



Keep Dry




Do not reuse



Keep away from
Sunlight



Consult instructions
for use

 **Sentiss Pharma Pvt. Ltd.**
Vill. Khera Nihla, Teh. Nalagarh,
Dist. Solan (HP)-174101, INDIA

Mfg. Lic. No.: MB/06/470


2460

EC REP

OBELIS S.A
Boulevard General Wahis 53
1030 Brussels, BELGIUM
Tel: +(32) 2.732.59.54
Fax: +(32) 2.732.60.03
E-Mail: mail@obelis.net

Imported & Distributed by:



BLUMONT



Blumont Healthcare Limited.
23 Moortown Close, Grantham,
NG31 9GG, United Kingdom

IFU1723.00